

AED INSTRUCTION

IPAD

1. SET THE DEFIBRILLATOR FOR CHILDREN (UNDER 8 YEARS/25 KG) OR FOR ADULTS (OVER 8 YEARS/25 KR) ON THE CONTACT AT THE TOP LEFT CORNER.
2. TURN ON THE DEFIBRILLATOR AT THE GREEN BUTTON MARKED WITH ①
3. THE ELECTRODES ARE PLACED AT THE BAG OF THE DEFIBRILLATOR IN A ROOM. PLACE THE ELECTRODES ACCORDING TO THE PICTURES ON THE NAKED AND DRY CHEST. IF MORE HELPERS ARE PRESENT, CONTINUE THE HEART PRESSURE MASSAGE WHILE PLACING THE ELECTRODES.
4. FOLLOW INSTRUCTIONS OF THE DEFIBRILLATOR. IF A SHOCK IS REQUIRED, THE BUTTON WITH THE ⚡-SYMBOL LIGHTS UP AND THEN PRESS THE BUTTON.

