

AED INSTRUCTION

LIFEPAK CR 2

1. AFTER OPENING THE COVER, THE DEFIBRILLATOR STARTS AUTOMATICALLY.
2. PLACE THE ELECTRODES ACCORDING TO THE PICTURES ON THE NAKED AND DRY CHEST. IF MORE HELPERS ARE PRESENT, CONTINUE THE HEART PRESSURE MASSAGE WHILE PLACING THE ELECTRODES.
3. FOLLOW THE INSTRUCTIONS OF THE DEFIBRILLATOR. IF A SHOCK IS REQUIRED, PRESS THE RED BUTTON WITH THE HEART SYMBOL. IN THE FULL AUTOMATIC VERSION, AUTOMATIC SUPPORT IS PROVIDED

