

AED INSTRUCTION PHILLIPS HEARTSTART

1. REMOVE THE COVER OF THE DEFIBRILLATOR AND PUT IT ASIDE, THE DEFIBRILLATOR STARTS AUTOMATICALLY.
ALTERNATIVELY, YOU CAN START THE DEFIBRILLATOR ON THE TURQUOISE BUTTON.
2. PLACE THE ELECTRODES ACCORDING TO THE PICTURES ON THE NAKED AND DRY CHEST. IF MORE HELPERS ARE PRESENT, CONTINUE THE HEART PRESSURE MASSAGE WHILE PLACING THE ELECTRODES.
3. FOLLOW INSTRUCTIONS OF THE DEFIBRILLATOR. IF A SHOCK IS REQUIRED, PRESS THE FLASHING RED BUTTON WITH THE ⚡SYMBOL.

